SK NÁVOD NA POUŽITIE

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VULM

EN INSTRUCTIONS FOR USE

Read the entire instructions for use carefully before using this medical device because it contains information that is useful for you.

RECTOVENAL® ACUTE

anorectal gel for haemorrhoids
medical device

RECTOVENAL® ACUTE packaging content

A packaging of RECTOVENAL® ACUTE contains anorectal gel intended for use in haemorrhoids, instructions for use and practical applicator designed to simplify the gel application. Not all pack sizes may be marketed. RECTOVENAL® ACUTE contains the oak bark extract (Quercus robur), diosmin, hesperidin, carbomer 940, sodium hydroxide, methylparaben, propylparaben and purified water.

Mechanism of action of RECTOVENAL® ACUTE

The positive effects of the RECTOVENAL® ACUTE medical device are based on the shrinking effects of the tannins contained in the oak bark extract (Quercus robur). By their shrinking and soothing effects, the tannins act favourably in skin and mucosa inflammations, facilitate the healing of wounds and relieve the symptoms accompanying the haemorrhoids occurrence. By their ability to form a protective barrier, the tannins protect the sensitive skin and mucosa in the anal area from possible infection and support their recovery and regeneration. The flavonoids diosmin and hesperidin support the favourable antioxidant, antibacterial and anti-inflammatory effects of the oak bark tannins.

Intended use of RECTOVENAL® ACUTE

The RECTOVENAL® ACUTE medical device is used to relieve the symptoms accompanying the haemorrhoids occurrence such as irritation, itching, burning or oozing in the anal area. At the same time, RECTOVENAL® ACUTE reduces the swelling that occurs in inflammatory processes and supports wounds healing. RECTOVENAL® ACUTE is intended for use in the anal area and must not be used internally. The application of RECTOVENAL® ACUTE in pregnancy or during lactation should be consulted with your physician. The application of RECTOVENAL® ACUTE to children or adolescents should be consulted with your physician.

How to use RECTOVENAL® ACUTE

As needed, sufficient amount of the gel should be applied in to the anal area 2 – 3 times a day. It is recommended that before the application, the skin and mucosa around the anal area should be cleaned with warm water (without the use of soap) and dried with a soft towel. Before each application, remove the gel remains after the prior application of the gel or other locally used products remains.

RECTOVENAL® ACUTE is usually used in the morning and before going to bed in the evening. It is recommended that the gel should be used 2 – 3 times a day for 5 – 7 days; then 2 – 3 times per week until the symptoms disappear. The gel application must not exceed 30 days.

RECTOVENAL® ACUTE may be applied in the anal area in following two ways: 1) direct application using your fingers or, 2) use of the enclosed applicator.

Direct gel application using your fingers:

Squeeze the plastic tube to deliver a sufficient amount of the gel directly on your fingers and use the fingers to apply the gel carefully around the anal area. Screw the cap on to the tube immediately after gel application to prevent the gel from drying.

The gel application with the enclosed applicator:

After you remove the cap from the tube, screw the enclosed applicator on to the tube and remove the applicator's upper cover. Place the tube with the applicator in to the anal area and squeeze the tube gently until a sufficient amount of the gel is delivered. Immediately after the gel application, unscrew the applicator and wash it with warm water. When the applicator dries up, fix the upper cover on to it. Close the tube with the cap in order to prevent the gel from drying.

Important warnings and special instructions before the use of RECTOVENAL® ACUTE

RECTOVENAL® ACUTE is only intended for external use in the anal area and must not be used internally. While using RECTOVENAL® ACUTE, you must observe the manner and conditions of use that are specified in these instructions for use.

Do not use RECTOVENAL® ACUTE if you are hypersensitive (allergic) to any component contained in this medical device. The preservatives methylparaben and propylparaben contained in RECTOVENAL® ACUTE may cause allergic reactions (such hypersensitive reactions may even be delayed).

At the occurrence of bleeding in anal area or suspicion of blood in stool accompanied by fever and pain, do not continue using RECTOVENAL® ACUTE and consult a further use of the gel with your physician. If the problems with haemorrhoids last more than 30 days, further use of the gel should be consulted with your physician.

The concurrent use of RECTOVENAL® ACUTE and other locally applied products (creams, ointments, gels or suppositories) should be consulted with your physician.

Do not use RECTOVENAL® ACUTE after the expiry date stated on the packaging. Do not use RECTOVENAL® ACUTE if you notice the tube is damaged to the extent that the upper cover does not hold tight or the gel is leaking. Return the unused RECTOVENAL® ACUTE medical device to your pharmacy for safe disposal.

Possible side effects of RECTOVENAL® ACUTE

During use of RECTOVENAL® ACUTE side effects may occur having the form of hypersensitivity (allergy) or local skin or mucosa irritation in the anal area (contact dermatitis).

In the event that you observe the occurrence of any type of side effect including one not described in these instructions for use, refer to your physician.

Storage conditions of RECTOVENAL® ACUTE

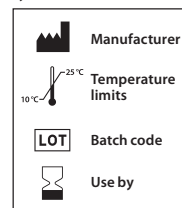
RECTOVENAL® ACUTE has to be stored in the original tube properly closed, at the temperatures between 10 °C and 25 °C, in a dry place, protected from frost, direct solar radiation and radiant heat. Keep out of the sight and reach of children.

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